

ToxTilt

Urine Drug Test

Make urine drug testing easy with ToxTilt!

The Swift Toxicology ToxTilt Urine Drug Test cup is an all-in-one solution for accurate urine drug testing. The instructions on the kits are easy to follow and can detect six common drug classes as required under AS/NZS 4308:2023, including amphetamines, benzodiazepines, cocaine, methamphetamines, opiates, and THC (marijuana).

The ease of use and instant on-site results make it perfect for workplaces, pre-employment screening, and personal use.

The ToxTilt urine drug test cup is verified for compliance with all drug groups stipulated in AS/NZS 4308:2023 (per appendix B4) and creatinine (per appendix B3). ToxTilt also features an integrated temperature strip and adulterant test strips to help ensure the sample has not been adulterated.

Independent laboratory verification reports / certificates of compliance are available on request.



Ordering Information:

52301 ToxTilt Urine Drug Test Cup (6 Drugs + 6 Adulterants)
Tests for: AMP (300) / BZO (200) / COC (150) / MET (300) / OPI (300) / THC (50)
Adulterants: NIT / GLUT / CRE / OX / pH / SG

Custom test configurations are available on request.

1



Collect fresh urine sample above minimum fill line.

2




Secure lid tightly until snug.

3

READ	90	92	94	96	98	100
GREEN			●			
COLOR	32	33	34	36	37	38


Verify temperature is within range.

4

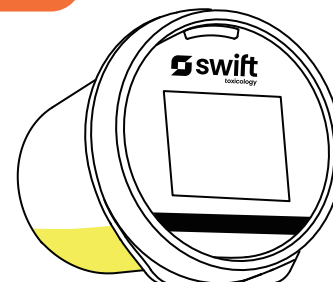

Tilt the cup on its side to start the test and leave on flat surface.

5

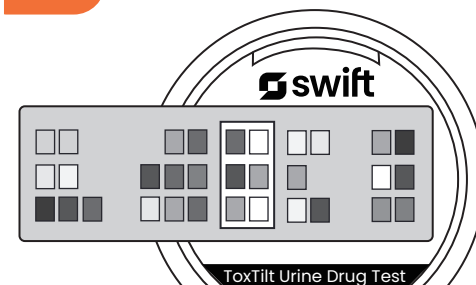

Read the adulteration test results at 3-5 mins using the colour card.

6

Read negative results as soon as all test (T) and control (C) lines appear. Wait 5 minutes before declaring not-negative.

NEGATIVE

All Control and Test Lines Present

	OPI	COC	MET	AMP	BZO	THC
C	+	+	+	+	+	+
T	+	+	+	+	+	+

NEGATIVE

Do Not Compare Line Intensity

	OPI	COC	MET	AMP	BZO	THC
C	+	+	+	+	+	+
T	+	+	+	+	+	+

NOT-NEGATIVE

To THC & AMP

	OPI	COC	MET	AMP	BZO	THC
C	+	+	+	+	+	+
T	+	+	+	+	+	+

NOT-NEGATIVE

To MET & BZO

	OPI	COC	MET	AMP	BZO	THC
C	+	+	+	+	+	+
T	+	+	+	+	+	+

INVALID

No "C" Line Present

	OPI	COC	MET	AMP	BZO	THC
C	+	+	+	+	+	+
T	+	+	+	+	+	+

INVALID

No "C" Line Present

	OPI	COC	MET	AMP	BZO	THC
C	+	+	+	+	+	+
T	+	+	+	+	+	+

Please refer to Instructions for Use for detailed product information.